

IDEXX VetLab[®] Station



**Laboratory Information Management
System – for the Integrated Practice**

The IDEXX VetLab[®] Station brings it

Start here!

1. Press “Analyse Sample” and enter your patient details
2. Select the analysers you want and press “Run”



3. Run your IDEXX VetLab[®] in-house analysers



4. Collect your printed results. If you would like to see a trending, go to “Records”, select your patient and the test you want to trend



Client: Adams, Kimberly (32788) Gender: Female/S
Patient Name: Charley Weight: 4.3 kgs
Species: Feline
Breed: Mixed

Test Results

Catalyst Dx (April 22, 2014)

GLU	11.8 mmol/L
BUN	10 mmol/L
CREA	141 µmol/L
BUN/CREA	1.3
PHOS	1.68 mmol/L
Ca	2.53 mmol/L
TP	67 g/L
ALB	30 g/L
GLOB	37 g/L
ALB/GLOB	0.8
ALT	48 U/L
ALP	39 U/L
GGT	7 U/L
TBL	2 µmol/L
CHOL	4.86 mmol/L
NAN	40
Osm Calc	333 mmol/L

VetStat (April 22, 2014)

pH(wet)	7.48
PCO2(wet)	19.9 mmHg
PO2(wet)	29.0 mmHg
ArGap	28 mmHg
CO2	20.0 mmHg
Na	154.0 mmol/L
K	4.7 mmol/L
Cl	113.0 mmol/L

Client: Adams, Kimberly (32788) Gender: Female/S

Patient Name: Charley Weight: 4.3 kgs

Species: Feline

Breed: Mixed

Test Results

ProCyt Dx (April 22, 2014)

RBC	8.26 x 10 ¹² /L
HCT	45.0%
HGB	12.5 g/dL
MCH	46.6 fL
MCHC	14.50 g/dL
RDW	19.8%
%RETIC	0.5%
RETIC	14.8 x 10 ⁹ /L
WBC	16.42 x 10 ⁹ /L
NEU	78.3%
%LYM	12.4%
%MONO	9.1%
%EOS	0.2%
%BASO	0.1%
NEU	14.79 x 10 ⁹ /L
BAND	* Suspect Pre
LYM	* 2.8 x 10 ⁹ /L
MONO	0.78 x 10 ⁹ /L
EOS	0.02 x 10 ⁹ /L
BASO	0.02 x 10 ⁹ /L
PLT	280 x 10 ⁹ /L
BAND	* Suspect Pre

Printed: April 23, 2014

Printed: April 23, 2014

Prior result

16/07/17

8.30 x 10¹²/L

36%

13 g/dL

42.7 fL

14.53 pg

34.0 g/dL

18.2 %

0.3 %

14.5 x 10⁹/L

14.28 x 10⁹/L

78.6 %

16.5 %

3.8 %

1.1 %

0.0 %

11.22 x 10⁹/L

Gender: Female (Spayed)
Weight: 4.3 kgs
Age: 9 Years
Doctor: Kairim, Graham

Kensington Vets
Kensington High Street

Kensington Vets
Kensington High Street
London

Reference Interval	LOW	NORMAL	HIGH
014 4:45 PM)			
16.01/14.9/42 AM			
8.30 x 10 ¹² /L			
36%			
13 g/dL			
42.7 fL			
14.53 pg			
34.0 g/dL			
18.2 %			
0.3 %			
14.5 x 10 ⁹ /L			
14.28 x 10 ⁹ /L			
78.6 %			
16.5 %			
3.8 %			
1.1 %			
0.0 %			
11.22 x 10 ⁹ /L			

RBC Run

WBC Run

Fluorescence

Granularity

PLT

NEU

EOS

LYM

BASO

MONO

UREC

52 AM

Page 1 of 2

idexx
LABORATORIES

Integrated report

- View a comprehensive record of each patient's diagnostic history
- See prior results in context with current results for easy comparison plus results in organ-specific order to easily detect abnormalities
- Quickly determine disease status and change
- Ensure every test result is associated with that patient's medical record

- Control your data by integrating medical, patient and client information into one record
- Run all your IDEXX VetLab® analysers from one screen
- Reduce data entry errors with autofilling of client and patient information
- Access integrated reports and virtually unlimited storage capacity
- Gain compliance through better record keeping
- Benefit from the support of IDEXX SmartService™*

- Consolidate results from all your IDEXX VetLab® analysers, including SNAP® test results in one report
- Assess disease progression through graphing and trending
- Improve client communications with diagnostic or treatment progression printout
- Link to expert descriptions and common causes of abnormal values

* For more information on IDEXX SmartService™ please visit <http://www.idexx.nl/diergezondheid/analysatoren/smartservice/>

The advantages of the Integrated Practice

Connect to practice management software* for completely automated information management and watch your bottom line improve

- Rely on accurate and complete billing and reduce missed charges with automatic downloads to patient files and invoices
- Automatically populate diagnostic requisitions and upload results to electronic medical records

“ The benefits of in-house diagnostic testing, especially with an integrated testing system, are real. With minimal effort, veterinarians can utilise the tools they need to drive practice growth and practise a level of medicine that provides convenience to clients and compassion to patients.** ”

Fred Metzger, DVM, DABVP



* Contact IDEXX for a list of compatible practice management softwares

** Metzger, F. Integrated in-house diagnostics: Better medicine, better management, *Diagnostic Insights*, 2007

The IDEXX **VetLab® Suite** is just one way we support veterinary practices like yours across the globe. Our integrated system of products and services is designed to save you time and improve your practice's productivity – simply and seamlessly.



SOWAR PRIVATE LIMITED

CSC, C/9, Vasant Kunj, New Delhi-110070

Tel: +91-11-4176 7519-22

Fax: +91-11-2612 2809

E-mail: sowar@sowarmedical.com

www.sowarmedical.com